

VENOSAN®

Compression Stockings

Instructions for use Compression Armsleeves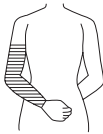


CG

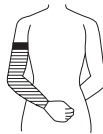
CG-H

CH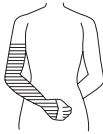

ZG

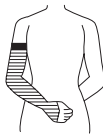ZG-H

ZH

Instructions for use

Your VENOSAN® armsleeve has been produced in accordance with the latest medical knowledge and state-of-the-art textile technology.

It contains ecological fibres (tested in accordance with the "Oeko-Tex standard 100"), which will improve your quality of life and sense of well-being. The VENOSAN® armsleeve is a medical aid, and it is therefore absolutely essential that you read these instructions very carefully.

Important

- Always wait until the wound has been treated before putting on the medical armsleeve.
- The medical armsleeve must never be out on or worn over an open wound.
- Sharp fingernails and rings with sharp edges can easily damage the VENOSAN® armsleeve. Pay attention to these risks when putting the armsleeve on and removing it.
- It is recommended to use VENOSAN® gloves for donning and doffing.
- Do not pull or cut any loose ends you may find on the inside of the armsleeve.
- Make sure that the armsleeve does not come into contact with pointments or oil (including body lotion) and avoid exposure to direct sunlight, strong light and excessive heat. All these factors can ruin the armsleeve.
- Certain creams, ointments and oils can destroy your armsleeve or leave unsightly stains.

If you use and care for the armsleeve according to the instructions it will have a useful life of 6 months.

Care / Washing

- The VENOSAN® armsleeve can be simply washed in the washing machine at any temperature up to 40° C and be treated like any other ordinary synthetic fibre underwear.
- Regular washing does not damage the armsleeve.
- Do not use any fabric softeners, optical brighteners or stain removers (such as turpentine, benzine etc.).
- Regularly clean the silicone band on self-supporting top with alcohol to improve adhesion.

Manifest lymphoedema requires complex, medically prescribed decongestion treatment. Regular follow-ups by a doctor are necessary to ensure continued success of the therapy. Consult your doctor or stocking specialist immediately if you feel unwell, have acute pain, swelling on arm/hand or skin irritation occurs while wearing the armsleeve.

General Indications

- for mild and still soft lymphoedema
- post-mastectomy congestion and swelling, whether followed or not by radiotherapy
- post-operative and post-traumatic oedema after completion of basic treatment
- polyarthritic thickening of tissues
- compression treatment for burns and extensive scarring
- prevention of reoccurrence of lymphoedema after successful decongestion therapy for pronounced swelling

Contraindications

- inflammatory processes.

Risks and side effects

The use of armsleeves with silicone self-supporting tops may lead to skin irritation in case of sensitive skins.

Storage

Store the armsleeve at room temperature and protect against humidity and direct sunlight.

No liability accepted in case of failure to observe instructions in the leaflet.

SWISSLASTIC AG ST. GALLEN • Rorschacher Str. 304 •
CH-9016 St. Gallen • Switzerland • www.venosan.com

VENOSAN® is a registered trademark of
SWISSLASTIC AG ST. GALLEN, Switzerland

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