**VENOSAN®**

Compression Stockings

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Your VENOSAN® stockings have been produced in accordance with the latest medical knowledge and state-of-the-art textile technology. They contain ecological fibres (tested in accordance with the «Oeko-Tex standard 100»), which will improve your quality of life and sense of well-being. Your VENOSAN® stockings comply with all currently applicable national and international regulations. They bear the quality symbol for medical compression stockings, and are tested periodically by official test organisations. Your VENOSAN® stockings are a medical device, and it is therefore absolutely essential that you read these instructions very carefully.

### **Important**

Wash before use! Be sure to consult trained medical personnel before wearing your stockings for the first time.

It is very important that the stockings are the correct size and fit you exactly.

Never fold over the top seam. This will double the pressure which can lead to dangerous constriction.

If the pain in your legs increases when you are wearing your medical compression stockings, or if you feel unwell, you must consult your doctor or a suitable specialist immediately.

Please make sure you wear suitable shoes that will not damage the sheer knitted fabric of your medical compression stockings.

Certain creams, ointments and oils can destroy your stockings or leave unsightly stains.

If you use and care for your stockings according to the instructions they will have a useful life of 6 months. After this period the medical efficacy can no longer be guaranteed, and we recommend you buy new stockings.

Please also see our website [www.venosan.com](http://www.venosan.com).

### **Putting the stockings on**

Sharp or cracked finger nails, rough skin, sharp-edged rings, bracelets, or wristwatches can damage the sheer knitted fabric of your VENOSAN® medical compression stockings when you are putting them on or taking them off. It is, therefore, advisable to wear perfectly ordinary kitchen gloves. Please proceed as follows:

1. Make sure that neither your feet nor your legs are wet or even damp. Talcum powder can be helpful if necessary.
2. Turn the leg of the stocking inside out down as far as the heel, i.e. except for the foot, the inside of your VENOSAN® stocking must be facing outwards.
3. Use both thumbs to stretch the foot of the stocking open and pull your VENOSAN® stocking over your foot.
4. Grip the upper edge of the stocking and pull it over your heel.
5. Once all the leg of the stocking is above your ankle, reach inside with both thumbs and massage the fabric up your leg with a zigzag movement.

Since your VENOSAN® stockings are made of highly elastic material it is very important that you do not pull the top seam up as hard as you can. The stocking has to be massaged upwards in a series of side-to-side movements as described under 5 above.

### **Taking the stockings off**

1. Pull the top seam of the leg part downwards as far as your ankle.
2. Push both thumbs between the knitted fabric and your leg and ease it over your heel.
3. Pull the stocking slowly over your foot.

## Washing / Care

You can simply wash your VENOSAN® stockings in the washing machine at any temperature up to 40 °C and treat them like any other ordinary synthetic fibre underwear. Regular washing will not damage your stockings. Do not use any fabric softeners, optical brighteners or stain removers (such as turpentine, benzine etc.).

Clean the silicone bands on self-supporting tops with alcohol regularly to improve adhesion.

## Indications

Primary and secondary varicosis, varicosis during pregnancy, after sclerotherapy, after vein surgery, thrombophlebitis (superficial) as well as after treatment of phlebitis, deep venous thrombosis, after thrombosis, post-thrombotic syndrome, prophylaxis of thrombosis for ambulatory patients, CVI grades I to III according to Widmer, C1S-C6 according to CEAP, prevention of venous ulcers, insufficiency of the communicating veins, angiodysplasia, lymphoedema, oedema during pregnancy, post-traumatic oedema, postoperative oedema, idiopathic cyclic oedema, grade II lipoedema and above, congestive conditions following prolonged immobility (arthrogenic stasis syndrome, paralysis and partial paralysis of the limb).

Further indications: After burns, hypertrophic scarring

## Contraindications

VENOSAN® medical compression stockings must not be worn by anyone suffering from the following:

Absolute:

- severe arterial circulatory disorders
- severe neuropathological disorders
- severe microangiopathological disorders
- by bedridden patients

Relative:

- very severe oedema
- weeping dermatitis
- decompensated heart failure
- immediately after deep venous thrombosis

Failure to observe these contraindications will result in the invalidation of any liability claims.

## Risks and side effects

The use of stockings with silicone self-supporting tops may lead to skin irritation in case of sensitive skin.

## Storage

Store the stockings at room temperature and protect against humidity and direct sunlight.

## 100% Quality guarantee

Your VENOSAN® stockings were carefully checked before they left the factory. If any fault in the knit should appear within six months, despite your having looked after them properly, we will replace the stockings free of charge. Please return the faulty VENOSAN® stockings to the dealer you bought them from, with the receipt.

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VENOSAN® is a registered trademark of SWISSLASTIC AG ST. GALLEN, Switzerland
SWISSLASTIC AG ST. GALLEN